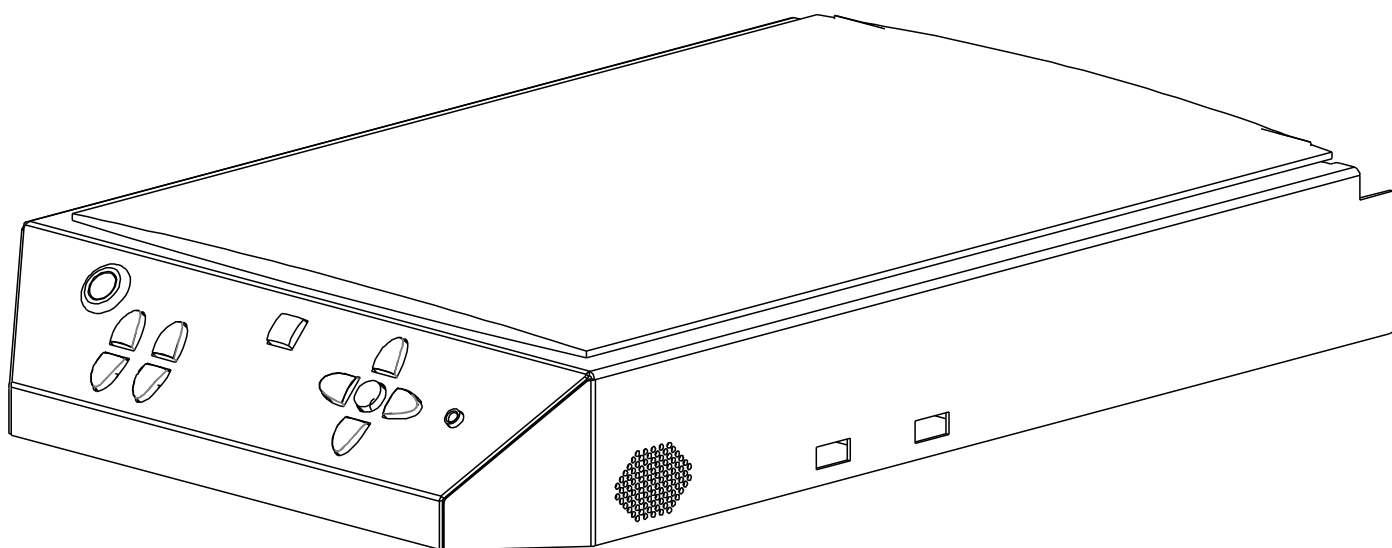
---

# EasyReader

*Model: EasyReader 2012*

*Serial Number: ER00845 and up*

---

**Français - Mode d'emploi- Page 2**  
**Nederlands – Gebruikshandleiding - Pagina 10**  
**English - User Manual - Page 18**  
**Deutsch – Gebrauchsanweisung - Seite 26**  
**Specifications 34**

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# English – User Manual

This manual describes the following versions of the KOBA Vision EasyReader. The latest versions of the manual can be downloaded from the KOBA Vision website: <http://www.kobavision.be>.

Release date: July, 6<sup>th</sup>, 2015.

## Chapter 1: Safety and maintenance instructions

### SAFETY INSTRUCTIONS

- Text read by the EasyReader is under no circumstance guaranteed to be a 100% correct. The text-to-speech function is solely to be used as a reading aid.
- The device is not to be used in a medical environment.
- Save this manual carefully. It contains all necessary information about your device.
- Keep the device in a dry, well-vented and clean spot. Do not place the device in direct sunlight or next to a source of heat.
- Never use this device in ambient temperatures higher than 35°C.
- This device is designed as aid to read aloud documents. Any change of purpose voids the CE-mark.
- The connections have been designed to only connect in 1 way. Never force a connection. Install the power supply of the device in a well ventilated area.
- Never attempt to repair the device by yourself. It is dangerous to remove protective covers which are there to prevent electric shock. You must contact KOBA Vision or your dealer in case of defects.
- To avoid electrocution, do not place the device in a spot where it can come into contact with liquids.
- Your product is designed and manufactured with high quality materials and components, which can be recycled and reused. Do not dispose of your old product in your general household waste bin. Dispose of the complete product (including its cables, plugs and accessories) in the designated collection facilities.

### MAINTENANCE INSTRUCTIONS

- The device can be cleaned with a damp cloth. Never use chemicals for they can damage the finish and/or other parts.
- Occasionally clean the glass plate of the scanner with a damp cloth.
- The adapter must only be replaced by the same type (15V, max. 2,5A, EN60601-1) after switching off the device and removing the adapter from the grid.
- Treat your device with care and you will enjoy it for years to come.

## ***Chapter 2: First-time use***

You need to go through the following steps to use the EasyReader for the first time:

### **INSTALLATION OF THE BATTERY**

Follow the steps to install the battery:

1. **Unscrew the 2 hand screws at the back of the device and put of the cover.**
5. **Put the battery in the sleeve.**
6. **Screw the 2 hand screws with the cover.**
7. Connect the pin of the adapter at the back of the EasyReader. Then connect the reading device to the mains supply.

### **FIRST START UP**

After the device has been installed, you test whether it works. You switch on the EasyReader. (see next chapter – Switching on/off)

When starting for the first time you will see the language menu. Please read “Adding voices or languages and adjusting the brightness” in this manual for further instructions.

## ***Chapter 3: General operation***

After having successfully connected and tested the EasyReader, we will now go into the controls and the operation of the EasyReader.

### **SWITCHING ON/OFF**

Switch on the reader by briefly pressing the red on/off button. This red round button is at the far left (**button 1**). Soon after, you will hear a beep that confirms it is switched on. As soon as the reader reports that it is ready for use, you can start.

In order to switch off the reader press the red on/off button again (**button 1**). The reader then reports the battery status. Pressing a second time switches the reader off.

### **VOLUME AND READING SPEED**

Just to the right below the red on/off button there are keys for the volume and reading speed. They are in a group of four buttons. In order to adjust the volume, press briefly on one of the blue arrow keys to the left. The top key increases the volume (**button 2**) and the bottom (**button 3**) key reduces the volume. In order to change the reading speed, press briefly on one of the other two blue arrow keys. These keys are to the right of the volume keys. You increase the speed with the top key (**button 4**) and reduce the reading speed with the bottom key (**button 5**).

### **PITCH CONTROL**

Press long on the top reading speed button (**button 4**) to increase pitch.

Press long on the bottom reading speed button (**button 5**) to decrease pitch.

# SCANNING

In order to read out a text you must first scan it. Put the page on the glass plate with the text facing downwards. Then press briefly on the scan button (**button 6**). This is the blue square button at the top in the middle. After a short while the reader begins to read out the scanned text.

## SCANNING METHODS

In normal mode the reader will automatically use column recognition, and the columns will be read starting from the top-left column, after which the reader will navigate to the right column by column. When reading invoices and bank statements, this will sometimes not yield the desired result. If you press the scan key, the square button in the middle of the reader, for a while then the text will be scanned again but this time without column recognition. Remember that the lines will now be read over the page width.

## PAGE ORIENTATION

Press long on the lower navigation button (**button 9**) to consult the page orientation. This can only be done after performing a scan.

## NAVIGATION

With the five blue keys in the shape of a cross, you can navigate or browse through the text. These arrows are grouped on the right-hand side. With the round key in the middle (**button 7**) you can start or pause the reading out. With the left arrow key you go backwards through the text (**button 10**), with the right arrow key forwards (**button 11**). With the top and bottom arrow key (**button 8 or 9**) you select the various navigation options. The various options are as follows:

- **Spelling:** The reader navigates by the letter. For spaces between words you hear nothing.
- **Word by word:** The reader navigates by the word.
- **Sentence by sentence:** The reader navigates by the sentence. A sentence is determined by full stops. The reader thus reads from full stop to full stop. This means that the reader may stop in the middle of a sentence, when there is an abbreviation with a point after it for example.
- **Line by line:** The reader navigates by the line. A line is a series of words and sentences on the same line from left to right in a text.
- **Block reading:** The reader navigates by the block. A block is a part of the text that is considered as a unit by the reader. It can be a title, a column, a sub-column or another text block.
- **Page by page:** The reader navigates by the page. Thus, you can consult the previous scanned page. In order to repeat the same page, you must first select page reading by the page with the top or bottom button and then press on the middle button for a while in order to repeat the page.

By pressing the middle button (**button 7**) for a while you can repeat the current word, sentence, etc.

## A FEW COMMENTS

- You always get spoken feedback when you press any key. However, when reading out no help messages are spoken.
- You can also connect headphones. The input for this is on the right-hand side of the operating panel (**plug 12**).
- Depending on the grey values, text in drawings or photos will not be read.
- You must flatten out crinkled pages. If necessary place a magazine or book on the page to press it flat.
- When using thick books the cover can be left open. It is only necessary to close the cover if there is too much incident light or the page is crinkled.
- You can scan a new page while the previous one is still being read out.
- If you switch off the reader, the last text is saved. This is immediately available when you switch the reader on again, and this by pressing the round key in the middle (**button 7**) of the navigation keys. If you scan a new page after starting, then the previous texts are deleted.
- If a number of languages are installed, you can change the language by pressing on one of the volume keys (**button 2 or 3**) for a while.
- Text in columns is read properly by the reader, except if the title of the article is in the middle of the text, for example, instead of the top. If when reading a sighted person has to look for the continuation of the text, a reading machine will often make mistakes.
- If you connect an USB stick or MP3 player to the reader, the reader will ask to save it. If you confirm, a text, MP3 file (spoken text) and a photo is saved of all the scanned texts. You can use this feature to listen to the text elsewhere or to save a text or photo on a PC.
- You can erase the memory on the reader by pressing the selection buttons for the navigation mode together long, i.e. the top and bottom arrow. (**button 8 and 9**)

## AUTOMATIC LANGUAGE RECOGNITION

You can activate the option “automatic language recognition” by pressing and holding one of the volume buttons (**button 2 or 3**) long until the EasyReader announces “automatic”.

Only languages selected in the languages menu will be recognized.

When this function is set, each text bloc will be read in the according language. The default voice is predefined. If you want the text to be read in another voice, only select 1 voice per language (your preferred voice) in the language menu.

## ADDING VOICES OR LANGUAGES AND ADJUSTING THE BRIGHTNESS

Your reader contains many languages, and also different voices for most languages. You can change them by pressing on both volume buttons at the same time. Afterwards in this menu you can also adjust the brightness of the operating buttons. This is done as follows:

Press on both volume buttons for around 10 seconds. You then enter the menu. This menu consists of four steps.

- **Step 1:** Set the language for the operating buttons. Your mother tongue is the right choice for this. You select a language or voice with the arrow keys. By pressing on an arrow key for a while you go to the next step.

- **Step 2:** Set the voice for the operating buttons. You can only choose from the voices of the language that you selected in step 1. You select a voice with the arrow keys. By pressing on an arrow key for a while you go to the next step.
- **Step 3:** Select extra reading voices. These are the languages and their voices that can be selected for reading out. You can select other languages or voices by pressing the arrow keys. In order to activate or deactivate a voice, press the square scan key. To confirm your choice, press on an arrow key for a while.
- **Step 4:** Now you come to the menu to adjust the brightness of the operating buttons. You adjust the brightness with the arrow buttons. To leave the menu press an arrow key for a while.

## ***Chapter 4: Optional keypad for storage***

If you have purchased the separate keypad, you can use it to save texts on the hard disk, an MP3 player, or a USB stick. The photo of the text is also saved in black-and-white.

At the top left is the switch for selecting the medium. With this you can select the hard disk, the MP3 player or the USB stick. Top right there are two arrow keys with which you can browse through the groups on the hard disk, MP3 player or USB stick. Each group contains 10 positions that you can liken to a folder on the computer. The 12 round buttons (4 columns and 3 rows) represent these positions. Each position is indicated by a number.

In order to read a text that is entered at a position, press briefly on the key. Then press on the same key to confirm or on another key to cancel. In order to save a text, press on the key for a while. Press again to confirm. Press on another key to cancel.

In order to delete a text, replace the current text with a new text which overwrites the old one.

## ***Chapter 5: Camera***

The following items are also supplied: a camera with connection cable, a page position accessory, a navigation block and a number of plastic sheets.

You proceed as follows. Connect the camera with the USB cable supplied. Soon after the reader reports that a camera is connected. The camera now replaces the scanner. Then place the camera in the page position accessory. Ensure that the legs of the accessory point in the same direction as the unfolded camera.

In order to read the page with the camera, lay it flat in the accessory with the facing text up. Press on the square scan button (**button 6**) in the middle of the keypad in order to read out the page.

With the navigation block you can determine the start position simply by placing the block somewhere on the text. The text that is read out depends on the navigation mode. Generally the reader is set for block reading so that the reader reads the text block under the navigation block. If you only want to read the word, then set the reader to word by word reading and position the block. If you put the block on a photo or white space, then an audio signal is given. You then have to move the block.

Note! If the navigation block is on the page then the reader will always navigate on the basis of the block. To continue reading or to use the back and next button on the reader you must always delete the block from the page.

Although the camera is faster, it often gives a less accurate result due to variable incident light and poorer page positioning.

# ***Chapter 6: Warranty***

## **EXPRESS WARRANTY**

KOBA Vision offers a two-year warranty on every KOBA VISION EasyReader, starting from the delivery date. The battery has a warranty of 6 months.

Warranty services are provided by KOBA Vision or a dealer acting for KOBA Vision. When the video magnifier cannot be repaired correctly, KOBA Vision can decide on its own initiative to replace the reading device by another video magnifier featuring the same specifications. All components and products replaced within the warranty period become property of KOBA Vision.

## **WARRANTY PROVISIONS**

The warranty becomes invalid if the video magnifier:

- Was not used according to instructions or was misused,
- Was damaged as a result of a fall or blow,
- Was disassembled or tinkered with,
- Was repaired earlier by a non-official service partner.

Consequential loss is under no circumstances covered by the warranty.

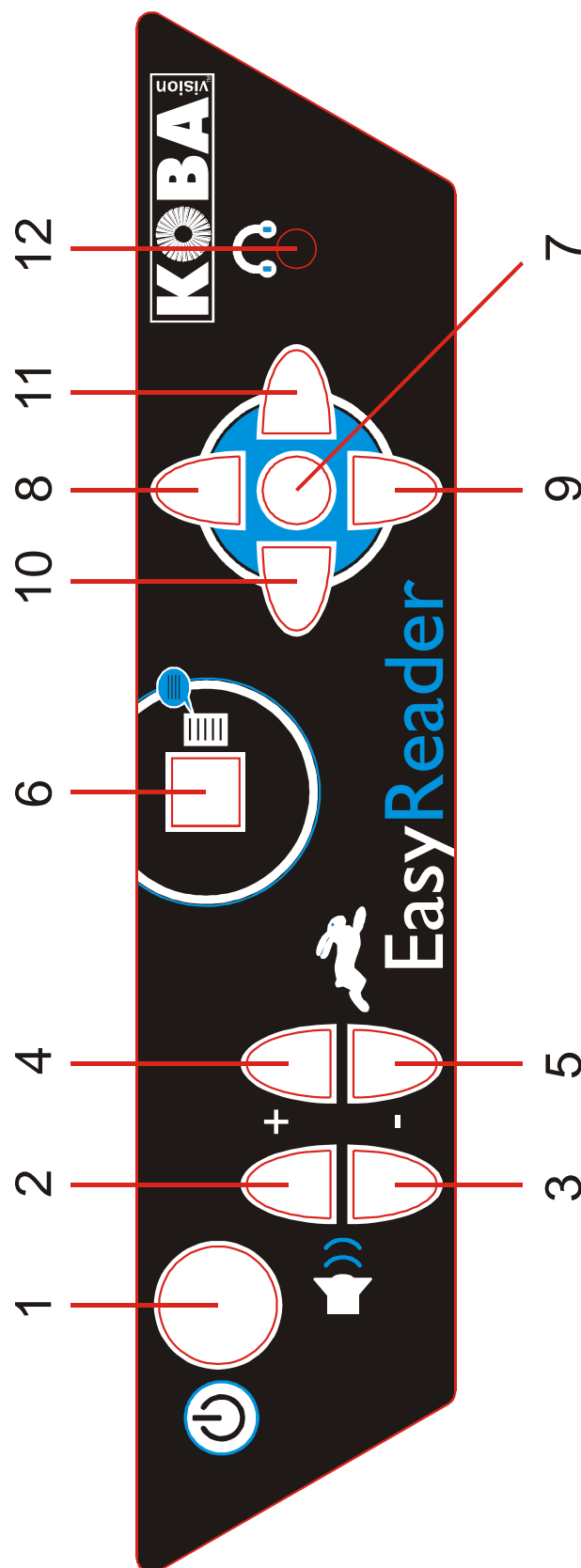
KOBA Vision bvba will neither assume any liability whatsoever nor accept any possibility of liability except for the warranty provisions stated above. This limitation to the liability does not imply a breach of the customer's legal consumer rights or his or her rights vis-à-vis the seller.

## **SERVICE FULFILMENT**

Before claiming the warranty or service and before contacting KOBA Vision bvba or a dealer, you must write down product name, serial number, purchase date, invoice number and any error messages or particular problem aspects.

# Chapter 7: Operating panel

1	<b>ON/OFF</b> switch
2	<b>Press briefly:</b> Increase volume <b>Hold button:</b> Select another reading voice
3	<b>Press briefly:</b> Decrease volume <b>Hold button:</b> Select another reading voice
4	<b>Press briefly:</b> Increase reading speed <b>Hold button:</b> Increase pitch
5	<b>Press briefly:</b> Decrease reading speed <b>Hold button:</b> Decrease pitch
6	<b>Press briefly:</b> Scan page and initiate automatic reading <b>Hold button:</b> Scan page without column recognition
7	<b>Press briefly:</b> Start/pause <b>Hold button:</b> Repeat
8-9	<b>Navigation modes</b> <b>Hold both buttons:</b> Erase texts in the machine
9	<b>Hold button:</b> Page orientation
10	<b>Back</b>
11	<b>Forward</b>
12	<b>Headphone jack</b>

# Sample text

## Snow White and the Seven Dwarfs

Once upon a time in the middle of winter, when the flakes of snow were falling like feathers from the sky, a queen sat at a window sewing, and the frame of the window was made of black ebony. And whilst she was sewing and looking out of the window at the snow, she pricked her finger with the needle, and three drops of blood fell upon the snow. And the red looked pretty upon the white snow, and she thought to herself, would that I had a child as white as snow, as red as blood, and as black as the wood of the window-frame.

Soon after that she had a little daughter, who was as white as snow, and as red as blood, and her hair was as black as ebony, and she was therefore called little snow-white. And when the child was born, the queen died.

After a year had passed the king took to himself another wife. She was a beautiful woman, but proud and haughty, and she could not bear that anyone else should surpass her in beauty. She had a wonderful looking-glass, and when she stood in front of it and looked at herself in it, and said, looking-glass, looking-glass, on the wall, who in this land is the fairest of all.

The looking-glass answered, thou, o queen, art the fairest of all.

Then she was satisfied, for she knew that the looking-glass spoke the truth.

But snow-white was growing up, and grew more and more beautiful, and when she was seven years old she was as beautiful as the day, and more beautiful than the queen herself. And once when the queen asked her looking-glass, looking-glass, looking-glass, on the wall, who in this land is the fairest of all.

It answered, thou art fairer than all who are here, lady queen.

But more beautiful still is snow-white, as I ween.

Then the queen was shocked, and turned yellow and green with envy. From that hour, whenever she looked at snow-white, her heart heaved in her breast, she hated the girl so much.

And envy and pride grew higher and higher in her heart like a weed, so that she had no peace day or night. She called a huntsman, and said, take the child away into the forest. I will no longer have her in my sight. Kill her, and bring me back her lung and liver as a token. The huntsman obeyed, and took her away but when he had drawn his knife, and was about to pierce snow-white's innocent heart, she began to weep, and said, ah dear huntsman, leave me my life. I will run away into the wild forest, and never come home again.

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